Excavators and Periodontal Curettes & Scalers

Excavators, Universal Curettes, Soft Tissue Curettes, Gracey Curettes and Dental Scalers

en-English

Attention

The operator/user must carefully read and understand this manual thoroughly to keep the product performance durable and reliable for defined life expectancy. After opening the packaging, first of all it is necessary to check the component(s) against the standard configuration. Check that these are all present and in perfect condition.

Intended Use:

Periodontal curettes and scalers are intended to be used for the removal of supragingival and subgingival calculus, plaque, and bacterial deposits from tooth surfaces and periodontal pockets. They assist in periodontal cleaning and maintenance. Whereas Excavators are intended to be used for the removal of carious dentin etc.

The Instrument Specific Intended Use is given below:

Universal Curettes:

Intended to be used for the removal of deposits from all tooth surfaces, including anterior and posterior teeth, using dual cutting edges.

Soft Tissue Curettes:

Intended to be used for the removal of granulation tissue and soft debris from periodontal pockets and surgical sites.

Gracey Curettes:

These are area-specific curettes with a single cutting edge, optimized for deep scaling and root planning in specific regions.

Dental Scalers:

Intended to be used for removal of supragingival and subgingival plaque and calculus deposits with sharp, precision-engineered tips.

Excavator:

Intended to be used for the removal of carious dentin, soft decayed tissue, and temporary restorative materials during cavity preparation.

 Our Dental Instruments are intended to be used by qualified dental professionals only in a professional dental facility.

Patient population:

These instruments are intended for use on adult and pediatric patients requiring dental procedures involving the removal of plaque, calculus, or decayed tissue. The selection of appropriate instrument size and technique should be based on the patient's age, oral anatomy, and specific clinical needs.

For Available Configurations/Models please visit the Periodontics and Restoration sections in our Catalogue

Features

Periodontal Curettes (Universal, Soft Tissue, and Gracey Curettes):

- High-quality stainless steel construction ensures long-lasting durability and resistance to corrosion.
- Finely honed cutting edges and optimized blade angulation provide effective scaling, root planing, and soft tissue debridement.
- Ergonomic, lightweight handles enhance grip and control, minimizing hand fatigue.
- Designed for efficient and minimally invasive treatment with a variety of blade configurations tailored to different periodontal needs.
- Compatible with standard sterilization protocols to maintain hygiene and instrument longevity.

Dental Scalers:

- Sharp, precision-engineered tips effectively remove supragingival and subgingival calculus and plaque deposits.
- High-grade stainless steel or titanium construction offers excellent resistance to corrosion and wear.

- Textured, lightweight handles improve grip and control, ensuring precise maneuverability during procedures.
- Available in various tip shapes and sizes to target different regions, including interproximal and hard-to-reach areas.
- Single or dual cutting edges allow for efficient scaling while minimizing trauma to surrounding tissues.

Excavator:

- Made from high-grade, autoclavable stainless steel for durability, corrosion resistance, and long-term sharpness retention.
- Finely contoured spoon-shaped tips allow accurate removal of decayed dentin.
- Available in multiple shapes and sizes to suit different cavity preparations.
- Balanced shank-to-blade ratio ensures flexibility while maintaining structural integrity.
- Textured or silicone-molded handles provide a firm, comfortable grip for enhanced control.

Operating instructions

These instruments are non-sterile and must be cleaned and sterilized before first use.

Always check if the reprocessed instruments are in perfect condition and do not use the instruments if any kind of deterioration, damage or deformation is observed. Regularly check for dull or damaged tips and replace when necessary.

Use a firm but controlled grip to maintain precision and stability. Avoid excessive force on root surfaces to prevent unnecessary damage and minimize the risk of enamel and soft tissue trauma. Maintain proper angulation for optimal scaling efficiency and precise adaptation to the tooth surface.

For Curettes:

- Hold the instrument using a modified pen grasp with a stable fulcrum for precision and control.
- Maintain a blade-to-tooth angle of 60°–80° for efficient scaling and root planing.
- Use long exploratory strokes for root planing and short, firm strokes for calculus removal.
- Universal curettes are suitable for all tooth surfaces, while Gracey curettes are designed for specific areas.
- Soft tissue curettes should be used gently to remove granulation tissue while preserving healthy structures.
Securely tighten the matrix band around the tooth, ensuring proper adaptation.
- Ensure a stable fit to prevent movement during restoration placement.

For Scalers:

- Hold the instrument using a modified pen grasp for precise control and tactile sensitivity.
- Establish a stable fulcrum on an adjacent tooth or intraoral structure for accuracy and reduced hand fatigue.
- Maintain close contact between the working end and the tooth surface, ensuring proper adaptation.
- Use a blade-to-tooth angle of 70°–80° for sickle scalers and 45°–90° for hoe scalers for effective calculus removal.
- Use controlled pull strokes directed coronally, engaging the wrist or forearm for activation.
- Adjust pressure based on the type and extent of calculus deposits to avoid trauma.

For Excavators:

- Use gentle scooping motions to remove carious dentin.
- Maintain controlled pressure to avoid excessive removal of healthy tooth structure.

Adverse Event

Any serious incident that has occurred concerning the device should be reported to the manufacturer and to the respective competent authority of the Member State in which the incident has occurred.

Cleaning & Sterilization

Cleaning

 Clean the device with care after each application to avoid the drying of contaminants.

Initial Cleaning

1. Remove any visible debris from the instrument using moisten a cotton swab or cloth with isopropyl or ethyl alcohol and gently wiping the surface of the product.
2. Use a soft brush and enzymatic cleaner to remove contaminants thoroughly.

Manual Cleaning

3. Prepare a solution of neutral pH enzymatic detergent according to the manufacturer's instructions.
4. Immerse in a pre-soak enzymatic cleaner solution
5. Thoroughly clean all surfaces of the Instrument using a soft brush or cloth. Pay close attention to any crevices or hard-to-reach areas.
6. Rinse thoroughly under lukewarm running tap water for a minimum of 30 seconds to remove all traces of detergent.

Automated Cleaning

1. Always clean in an appropriate box or cassette and ensure that they are properly positioned within the unit.
2. Always follow the equipment manufacturer's instructions and ensure the instruments are compatible with the cleaning system.
3. Visually inspect for cleanliness, If any contamination is visible, repeat the cleaning steps until required cleanliness is achieved.
4. Dry with clean, lint free cloth or filtered compressed air until there is no visible moisture. Follow STEAM STERILIZATION PROCEDURE.

 Ultrasonic cleaning may be performed in accordance with hospital or facility protocols. Ensure proper rinsing and drying after the process to prevent residue buildup.

Sterilization

Note: Ensure that the instruments are completely dry before sterilization.

Steam Autoclaving

1. Place the instrument in an approved sterilization pouch or wrap.
2. Sterilize using a steam autoclave following a validated sterilization cycle according to ISO 17665-1 and your facility's protocols. A typical recommended cycle is 132-134°C (270-273°F) for 4 minutes.
3. Verify no residual moisture remains after sterilization to prevent corrosion and ensure longevity.
4. Allow the instruments to cool and dry completely before handling. Store in a clean, dry, and controlled environment until ready for use.
5. Follow the pouch/wrap manufacturer instructions for storage conditions and maximum storage time.

Important Notes

- Always Inspect the instruments for any signs of deterioration, breakage and bent etc. after each sterilization cycle. Discard if damage is observed.
- Always follow your facility's infection control protocols.
- Refer to your autoclave manufacturer's instructions for proper operation and maintenance. The Sterilization equipment must be validated by the hospital and or sterilization equipment manufacturers.
- The above-mentioned cleaning and sterilization guidelines, provided by manufacturer are intended as procedures compatible with specific materials. Sterilization must be performed according to the Hospital/Clinic approved protocol.
- The responsibility for ensuring proper sterilization of instruments lies with the user. The effectiveness of sterilization depends on validated cleaning, packaging, and sterilization procedures carried out at the facility.

Disposal

Our Excavators, Periodontal Currettes & Scalers: Universal Currettes, Soft Tissue Currettes, Gracey Currettes and Dental Scalers contain no hazardous materials. However, these must be cleaned/sterilized as per defined procedures before disposition. Please follow the local and national regulations or healthcare facilities' defined disposal and waste management policies.

Disclaimer

The products must be used, reprocessed, and maintained strictly in accordance with the instructions provided above. Any deviation from these guidelines by the dental professional or user is undertaken at the user's sole risk. The Manufacturer will not accept any requests for refunds or exchanges under warranty for products that have not been handled and reprocessed in compliance with these instructions.

Explanation of utilized symbols

	Medical Device Compliant with Regulation (EU) 2017/745
	Consult instructions for use
	Medical Device
	Product Code/Catalogue Number
	Model/Variant

	Lot Number
	The Device is supplied non-sterile.
	Sterilizable in a steam sterilizer (autoclave) at 132-134°C (270-273°F)
	Manufacturing Date
	Manufacturer
	Authorized representative in the European Community
	Importer
	Keep in a cool, dry place
	Caution, read instructions

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