Endodontic Instruments

Endodontic Explorers, Endodontic Spreaders and Endodontic Forceps

en-English

Attention

The operator/user must carefully read and understand this manual thoroughly to keep the product performance durable and reliable for defined life expectancy. After opening the packaging, first of all it is necessary to check the component(s) against the standard configuration. Check that these are all present and in perfect condition.

Intended Use:

These instruments are intended to assist in various stages of endodontic procedures, including canal exploration, obturation, material handling, and retrieval. They enhance precision and control during root canal treatments.

The Instrument Specific Intended Use along with their variants is given below:

Endodontic Explorers:

Endodontic explorers are intended to be used for examining root canals, locating orifices, identifying obstructions, and assessing canal patency during endodontic procedures.

Endodontic Spreaders:

Endodontic spreaders are intended to be used for the lateral compaction of gutta-percha during root canal obturation to create a dense seal.

Endodontic Forceps:

Endodontic forceps are used for grasping, handling, and retrieving small endodontic materials such as gutta-percha points, paper points, and broken instruments. They ensure precise manipulation of materials within the root canal.

Variants:

Endo File Forceps: For securely grasping and handling endodontic files and small materials during root canal procedures.

Steiglitz Endo File Forceps: For retrieving fractured endodontic instruments and small root canal materials.

Endo-Tweezers:

Endo-tweezers are intended to be used for providing precise handling and placement of small endodontic materials within the root canal.

 Our Endodontic instruments are intended to be used by qualified dental/healthcare professionals only in a professional dental/healthcare facility.

Patient population:

These instruments are intended to be used on all patient populations. However, the instrument type/variant must be selected as per patient keeping in view the examination/treatment requirement.

For Available Configurations/Models please visit the Endodontics section in our Catalogue

Features

Endodontic Explorers & Spreaders:

- Dual-ended stainless-steel tips provide accurate canal navigation and tactile feedback.
- High-grade stainless steel ensures durability and compliance with autoclave sterilization.
- Lightweight, anti-slip design reduces fatigue and ensures controlled force application.
- ISO-standard conical tip enables uniform lateral compaction of gutta-percha.
- Electropolished finish reduces debris adhesion for easy cleaning.

Endodontic Forceps:

- Fine, tapered, or serrated tips securely grip small objects like gutta-percha points and broken instruments.
- Ergonomic, angled jaws provide optimal access to confined spaces.

- Spring-loaded, tension-controlled mechanism allows smooth and consistent operation.

Endo-Tweezers:

- Ultra-fine, serrated tips provide superior grip on delicate materials.
- Anti-slip handles ensure precision and control.
- Made from high-quality stainless steel, resistant to corrosion and repeated sterilization.

Operating instructions

These instruments are non-sterile and must be cleaned and sterilized before first use.

Always check if the reprocessed instruments are in perfect condition and do not use the instruments if any kind of deterioration, damage or deformation is observed.

For forceps, ensure a stable grip to prevent slippage of materials.

For Endodontic Explorers & Spreaders:

- Select the appropriate instrument based on procedural needs.
- Use the explorer to locate canal orifices and assess patency.
- When using spreaders, apply controlled lateral pressure to compact gutta-percha.
- Avoid excessive force to prevent damage to root structures.

For Endodontic Forceps:

- Choose the appropriate forceps for material retrieval or handling.
- Position the forceps securely around the material.
- Maintain a stable grip while removing or placing endodontic materials.
- Avoid excessive force to prevent accidental dropping.

For Endo-Tweezers:

- Use tweezers to handle small endodontic materials with precision.
- Maintain a firm grip to prevent material slippage.

Adverse Event

Any serious incident that has occurred concerning the device should be reported to the manufacturer and to the respective competent authority of the Member State in which the incident has occurred.

Cleaning & Sterilization

Cleaning

 Clean the device with care after each application to avoid the drying of contaminants.

Initial Cleaning

1. Remove any visible debris from the instrument using moisten a cotton swab or cloth with isopropyl or ethyl alcohol and gently wiping the surface of the product.
2. Use a soft brush and enzymatic cleaner to remove contaminants thoroughly.

Manual Cleaning

3. Prepare a solution of neutral pH enzymatic detergent according to the manufacturer's instructions.
4. Immerse in a pre-soak enzymatic cleaner solution
5. Thoroughly clean all surfaces of the Instrument using a soft brush or cloth. Pay close attention to any crevices or hard-to-reach areas.
6. Rinse thoroughly under lukewarm running tap water for a minimum of 30 seconds to remove all traces of detergent.

Automated Cleaning

1. Always clean in an appropriate box or cassette and ensure that they are properly positioned within the unit.
2. Always follow the equipment manufacturer's instructions and ensure the instruments are compatible with the cleaning system.
3. Visually inspect for cleanliness, If any contamination is visible, repeat the cleaning steps until required cleanliness is achieved.
4. Dry with clean, lint free cloth or filtered compressed air until there is no visible moisture. Follow STEAM STERILIZATION PROCEDURE.

 Ultrasonic cleaning may be performed in accordance with hospital or facility protocols. Ensure proper rinsing and drying after the process to prevent residue buildup.

Sterilization

Note: Ensure that the instruments are completely dry before sterilization.

Steam Autoclaving

1. Place the instrument in an approved sterilization pouch or wrap.
2. Sterilize using a steam autoclave following a validated sterilization cycle according to ISO 17665-1 and your facility's protocols. A typical recommended cycle is 132-134°C (270-273°F) for 4 minutes.
3. Verify no residual moisture remains after sterilization to prevent corrosion and ensure longevity.
4. Ensure that forceps hinges are lubricated with an appropriate medical-grade lubricant after sterilization to maintain smooth operation.
5. Allow the instruments to cool and dry completely before handling. Store in a clean, dry, and controlled environment until ready for use.
6. Follow the pouch/wrap manufacturer instructions for storage conditions and maximum storage time.
7. For forceps, periodically check the hinge movement to ensure continued smooth function.

Important Notes

- Always Inspect the instruments for any signs of deterioration, breakage and bent etc. after each sterilization cycle. Discard if damage is observed.
- Always follow your facility's infection control protocols.
- Refer to your autoclave manufacturer's instructions for proper operation and maintenance. The Sterilization equipment must be validated by the hospital and or sterilization equipment manufacturers.
- The above-mentioned cleaning and sterilization guidelines, provided by manufacturers, are intended as procedures compatible with specific materials. Sterilization must be performed according to the Hospital/Clinic approved protocol.
- The responsibility for ensuring proper sterilization of instruments lies with the user. The effectiveness of sterilization depends on validated cleaning, packaging, and sterilization procedures carried out at the facility.

Disposal

Our Endodontic Instruments Endodontic Explorers, Endodontic Spreaders and Endodontic Forceps contain no hazardous materials. However, these must be cleaned/sterilized as per defined procedures before disposition. Please follow the local and national regulations or healthcare facilities' defined disposal and waste management policies.

Disclaimer

The products must be used, reprocessed, and maintained strictly in accordance with the instructions provided above. Any deviation from these guidelines by the dental professional or user is undertaken at the user's sole risk. The Manufacturer will not accept any requests for refunds or exchanges under warranty for products that have not been handled and reprocessed in compliance with these instructions.

Explanation of utilized symbols

	Medical Device Compliant with Regulation (EU) 2017/745
	Consult instructions for use
	Medical Device
	Product Code/Catalogue Number
	Model/Variant
	Lot Number
	The Device is supplied non-sterile.
	Sterilizable in a steam sterilizer (autoclave) at 132-134°C (270-273°F)
	Manufacturing Date
	Manufacturer
	Authorized representative in the European Community
	Importer

	Keep in a cool, dry place
	Caution, read instructions

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