da Sugrens rengøring & desinfektion

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
WSP 01-2207

POLO DENT®

Suction Disinfection CONCENTRATE

en Cleaning & disinfection of suction systems
nl Afzuigstroomreining & -desinfectie
fr Nettoyant & désinfectant des systèmes d'aspiration
de Sauganlagenreinigung & Desinfektion
pl Środek do czyszczenia i dezynfekcji ssaków
da Sugrens rengøring & desinfektion
sv Rengörings- och desinfektionsmedel för sugsystem
no Rense- og desinfeksjonsmiddel på aspirator
fi Imulaitteiden puhdistus- ja desinfiointiaine

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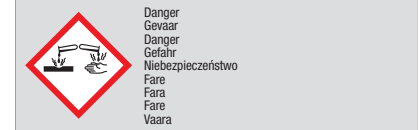EC REP DIRECTA AB
P.O. Box 723, Fimings väg 8 SE-194 27
Upplands Väsby, Sweden

CE
1639

en Cleaning & disinfection of suction systems

PoloDent Suction Disinfection • For the daily cleaning and maintenance of dental aspirator systems. Can be used on wet line, dry line and semi wet/dry line suction systems.

Directions for use, daily: Add 20ml ± 1ml of concentrate to 980ml warm water (2% solution). Aspirate via the high and low volume suction tubes. Leave for a minimum of 15 minutes to achieve full efficacy leave for 60 minutes) and then rinse thoroughly with clean water. For professional use only.

Efficacy: Bactericidal (15mins) including Salmonella typhimurium, Legionella pneumophila. Yeastcidal (15mins). Hepatitis B virus (15mins), Hepatitis C virus (15mins), Human Immunodeficiency Virus (15mins). Tuberculocidal (60mins). Fungicidal (30mins).

Tested to Standards: EN14885, EN14476, EN14348, EN13727, EN13697, EN13624, ASTM E1052.

Composition: Aqua, Dimethyldiethylammonium chloride, Dipropylene glycol, 2-Aminoethanol, Tetrasodium ethylene diamine tetracetate, Ethanol, Citric acid, Nonionic surfactant, Antifoam, Dye.

Danger:

H314: Causes severe skin burns and eye damage.
P260: Do not breathe dust/ fumes/gas/mist/vapours/spray.
P280: Wear protective gloves/protective clothing/eye protection/face protection.
P301+P330+P331: IF SWALLOWED: rinse mouth. Do NOT induce vomiting.
P303+P361+P353: IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water.
P304+P340: IF INHALED: Remove person to fresh air and keep comfortable for breathing.
P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Storage:

P403+P233: Store in a well-ventilated place. Keep container tightly closed.

P235: Keep cool.

Contraindications:

Incompatible materials: Aluminium, copper.

Disposal:

Do not discharge concentrate into drains or rivers.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

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