





Dispense on a clean, dry glass plate at approx. 20°C (68°F).

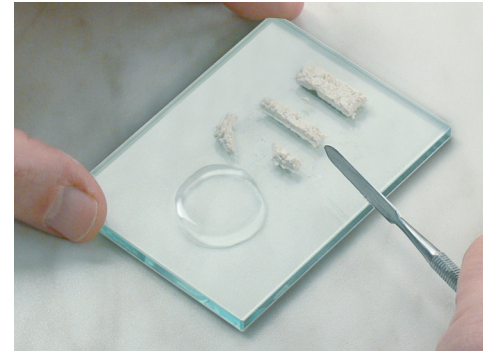

Divide into portions as follows:
 1/2 1/4 1/8 1/8.

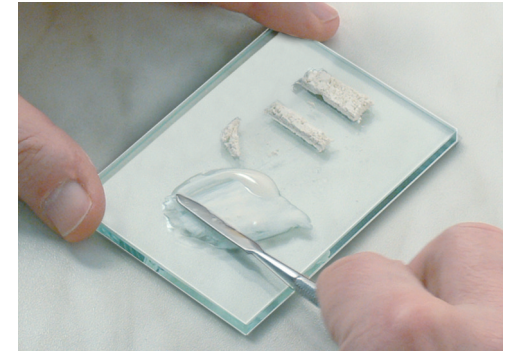

Mixing: Start with first 1/8 within 15 sec.

Harvard Cement Mixing Advice

Dispensing

Harvard Cement quick setting


- luting consistency
 powder (g) : liquid (g) = 1,8 : 1,0

Harvard Cement normal setting


- luting consistency
 powder (g) : liquid (g) = 1,5 : 1,0
- cavity lining consistency
 powder (g) : liquid (g) = 2,1 : 1,0



Second 1/8 within additional 15 sec.,
 mix with spreading movements.

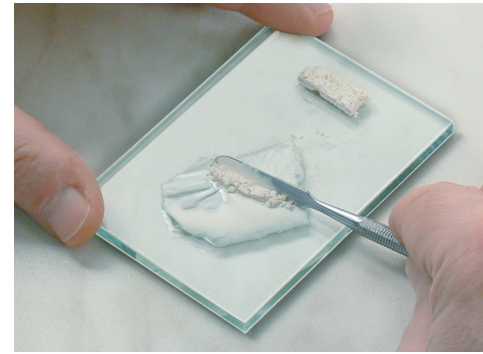


Draw 1/4 into the mixture.

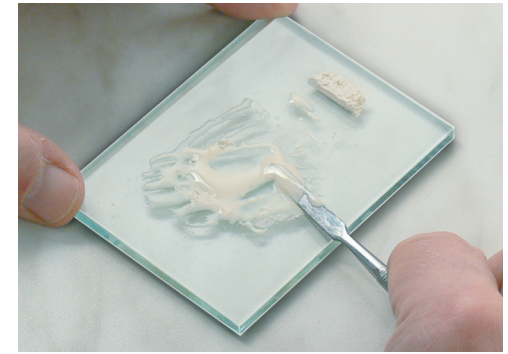

Mix with pressure and flat spatula within
 the next 30 sec.



Mix remaining 1/2 within further 30 sec.

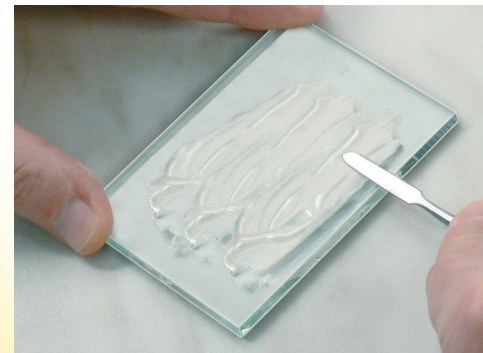

Use the entire surface of the glass plate.



Ready-for-use mixture within 90 sec.

