## "The original!"

CALASEPT<sup>®</sup> Calcium Hydroxide

### Design By Dentists<sup>™</sup>

Directa's products are developed and evaluated by our selected team of highlyqualified and renowned Swedish and international dental professionals with the aim of making life easier for the dental professionals in the daily clinical work.



**DIRECTA AB** P.O. Box 723, 194 27 Upplands Väsby, SWEDEN +46 8 506 505 75

DIRECTA INC 64 Barnabas Road, Unit 3 Newtown, CT 06470, United States 203.491.2273

2017-04

**DIRECTA GERMANY** Hammergutstr. 11 92245 Kümmersbruck, GERMANY

info@directadental.com www.directadental.com www.directadentalgroup.com www.facebook.com/directadentalgroup www.instagram.com/directadentalgroup www.twitter.com/D\_DentalGroup www.youtube.com/user/DirectaAB



CALASEPT Temp

## Calasept<sup>®</sup> Endoline

### **By DIRECTA**

Directa's products are developed and evaluated by our selected team of highly-qualified and renowned Swedish and international dentists with the aim of making life easier for the dentist in his daily clinical work.

### **Irrigation Solution**

Calasept<sup>®</sup> Sodium Calasept<sup>®</sup> EDTA Calasept<sup>®</sup> CHX

### **Irrigation Accessories**

### **Calcium Hydroxide**

Pure calcium hydroxide. High concentration of calcium hydroxid >41% giving a high concentration and release of hydroxyl ions.

Calasept<sup>®</sup> Calasept<sup>®</sup> Plus

### **Temporary Filling**

Self curing paste for a reliable closing of the cavity.

Calasept<sup>®</sup> Temp

### **Calasept® Endoline** effective treatment of root canals

Calasept is a well known brand of products from calcium hydroxide paste for temporary root fillings to a complete assortment of liquids for effective irrigation of root canals. It is a useful range of products for the proper treatment of root canals by any clinic, general practitioners and endodontic specialists.

Current products include Calasept and Calasept Plus calcium hydroxide paste, Calasept EDTA solution for chelation of root canals, Calasept Sodium Hypochlorite for the irrigation and debridement of the canals and Calasept CHX, a chlorhexidine for the final rinse of the canal. Our high quality Calasept Paste Needles and Double-Side-Vented Needles optimize the application and delivery of all Calasept products.

Our dedicated goal on the development and production of Calasept products has made the Calasept brand a reliable line where the prime focus is to facilitate and simplify the root canal treatment with innovative solutions for the dental practitioner.

### **Calasept®** Endoline

Disinfecting solution for different endodontic treatments.

Luer lock syringes for irrigation and luer lock irrigation needles.

Irrigation Syringes Calasept Irrigation Needles

### page: 4-9

page: 4-5 page: 6-7 page: 8-9

### page: 10-11

page: 10 page: 11

### page: 12-17

page: 12-13 page: 14-15

### page: 18-19

page: 18-19

#### 3



### Reliable endodontic irrigation

Effective against pathogenic organisms and pulp digestion. It is an excellent choice to optimize successful debridement during instrumentation thanks to its efficiency against pathogenic organisms and pulp digestion.

### For irrigation and debridement of root canals

Calasept Sodium Hypochlorite is used for the irrigation and debridement of root canals during endodontic treatment.


### **Antimicrobial and** tissue dissolving

Calasept Sodium is most reliable to your endodontic irrigation due to its antimicrobial and tissue dissolving activity Available in 3% and 6% solutions for a wider selection.

### Disinfecting solution

Used for irrigation and debridement of root canals and removal of smear layer and inorganic substances. Optimizes debridement during instrumentation.



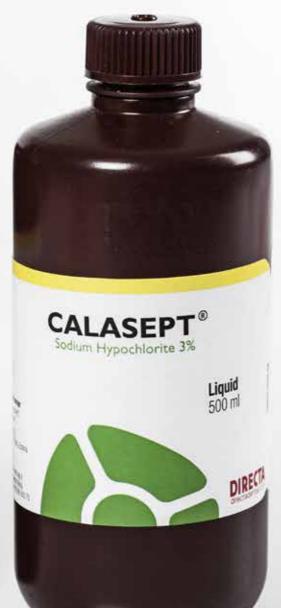

### **Calasept® Sodium Hypochlorite** disinfecting solution for endodontic irrigation during root canal treatment Prod No: 1290300 Calasept Sodium Hypochlorite 3%, 500 ml Prod No: 1290600 Calasept Sodium Hypochlorite 6%, 500 ml

Calasept Sodium Hypochlorite is used for the irrigation and debridement of root canals during endodontic treatment. Calasept Sodium Hypochlorite is most reliable to your endodontic irrigation due to its antimicrobial and tissue dissolving activity. It is an excellent choice to optimize successful debridement during instrumentation thanks to its efficiency against pathogenic organisms and pulp digestion.

Indications: Irrigation and debridement of root canals Optimizes debridement during instrumentation Removal of smear layer and inorganic substances

### reasons to use **Calasept<sup>®</sup> EDTA**

**Minimizes** file fractures Calasept EDTA has a Lubricating capacity.

### **Increased adhesion** to filling materials

Removes smear layer. The EDTA forms a water soluble chelate complex that decalcifies substances and dissolves smear layers, thereby increasing the adhesion of filling material.

### Easily flushed from the canal system

EDTA helps to enlarge and shape the canal for final obturation.

**Easy filling** of syringe The bottle neck with insert simplifies the filling of syringe.

CALASEPT ® EDTA EDTA Solution 17% DIRE

Adaptor for easy filling of syringe

Liquid 100 fi



# Calasept® EDTA irrigation agent for root canal walls chelating and smear layer removal

Prod No: 1260100 Calasept EDTA, 100 ml

Calasept EDTA is specially formulated to aid in the negotiation of root canals by endodontic instruments. The EDTA forms a water soluble chelate complex that decalcifies substances and dissolves smear layers, thereby increasing the adhesion of filling material. EDTA helps to enlarge and shape the canal for final obturation.

Rinse the root canal thoroughly after use of EDTA. Calasept CHX is recommended for final rinsing.







### reasons to use **Calasept<sup>®</sup> CHX**

Inhibitory **Capacity** Inhibits the growth of bacteria in root canals and dentin tubules.

**Final** rinsing Calasept CHX is recommended to use for final rinsing of the root canal after use of EDTA.

### Removing bacteria

Treatment using CHX 2% prior filling of the root canal material is more effective in removing bacteria.

**Easy filling Of Syringe** The bottle neck with insert simplifies the filling of syringe.



## Chlorhexidine Solution 2%



### **Calasept<sup>®</sup> CHX** for irrigation during endodontic treatment

Prod No: 1270100 Calasept CHX. 100 ml

Calasept CHX 2% solution is bactericidal. During endodontic treatment, EDTA and Sodium Hypochlorite are used intermittently. In vitro studies have established that the chelating effect of EDTA is not limited by using sodium hypochlorite. Therefore irrigation with Calasept CHX solution prior to filling a canal with root filling material is recommended. In addition, the bactericidal capacity of Calasept CHX will reduce the amount of bacteria.

\*NaCIO and CHX should not be mixed due to that an unwanted precipitate may occur.





### **Irrigation Syringes**

Prod No: 1210000 Irrigation Syringes, 20 pcs, 10 White & 10 Green

Luer lock syringes for irrigation in high quality, single use, 3 ml luer lock syringes. Colour-coded to eliminate the risk to mix up the irrrigation liquids.

Features: High quality 3-part syringe 3 ml Luer Lock Colour-coded

### Calasept<sup>®</sup> Irrigation Needles double side vent for efficient irrigation during root canal therapy

Prod No: 1201600 Calasept Irrigation Needle, Grey, 40 pcs needles, Container packed 27G Prod No: 1201610 Calasept Irrigation Needle, Violet, 40 pcs needles, Container packed 31G

With this high quality, double side vented, luer lock irrigation needle, you can be sure to get an effective and safe irrigation when performing an endo treatment. Easy and optimized to clean out the canals with the double side openings, creating a "swirl-effect". Safe with the rounded and closed tip, protecting apex.

Features: With luer lock hub Double side vented Bendable High quality stainless steel Sterile and disposable Easy use for cleaning out the canals Container packed

### reasons to use **Calasept**<sup>®</sup> Calcium Hydroxide

### Long-lasting and very effective

The very high concentration of >41% in combination with optimal calcium ion release generates the extremely efficient and long-lasting antimicrobial effect.

## **Direct, quick and**

**Calasept is a ready-to-use paste in air tight syringes for direct application through the sterilized needles. The air tight syringes eliminate the risk of incorporating air into calcium hydroxide turning the material into calcium carbonate.** 

### **Optimal** calcium release

Pure calcium hydroxide. High concentration of calcium hydroxide means long-lasting because the paste can release calcium ions for a long time.

### **Five indications**only one product

Can be used for temporary root-filling, pulp capping, pulp protection, insulation in deep cavities and for stepwise excavation.

### **Calasept**<sup>®</sup> **Calcium Hydroxide**

Prod No: 1230200 Calasept 2U, 2 syringes à 1,5 ml + 10 pcs Calasept Paste Needle Prod No: 1230100 Calasept 4U, 4 syringes à 1,5 ml + 20 pcs Calasept Paste Needle Prod No: 1231100 Calasept Paste Needle, 100 pcs Prod No: 690141 Grip Plate, 10 pcs

Calasept has a very high content of calcium hydroxide (> 41%) giving a high concentration of hydroxyl ions. These generate the high PH value of 12.4 which has a pronounced antimicrobial effect. High concentration of calcium hydroxide means long-lasting because the paste can release calcium ions for a long time. Calasept is a ready-to-use paste in air tight syringes for direct application through the sterilized needles.

Composition: Calcium hydroxide >41% Barium sulphate Sterile isotonic saline solution pH 12.4

### reasons to use **Calasept<sup>®</sup> Plus** Calcium Hydroxide

### Long-lasting and very effective

The very high concentration of >41%in combination with optimal calcium ion release generates the extremely efficient and long-lasting antimicrobial effect.

### **Smoother** consistency

The new production process gives Calasept Plus a consistency that allows calcium hydroxide to be applied using a small and flexible plastic needle, Flexi-Tip.

## **Precise and deep** application with Flexi-Tip

The autoclavable Flexi-Tip in combination with the smooth consistency is time saving and allows precise and deep application.

### **Five indications** only one product

With its multi purpose Calasept Plus can be used for Temporary root-filling, Pulp capping, Pulp protection, Insulation in deep cavities and for stepwise excavation.

### **Calasept<sup>®</sup> Plus Calcium Hydroxide**

Prod No: 124001 Calasept Plus Disposable, 3 Syringes à 0,4 ml + 3pcs Calasept Flexi-Tips Prod No: 1240000 Calasept Plus 1U, 1 syringe à 1,5 ml + 5 pcs Calasept Plus Flexi-Tips Prod No: 1240100 Calasept Plus 4U, 4 syringes à 1,5 ml + 20 pcs Calasept Plus Flexi-Tips Prod No: 1241100 Calasept Plus Flexi-Tip, 100 pcs Prod No: 690141 Grip Plate, 10 pcs

The consistency of new Calasept Plus allows calcium hydroxide to be applied using a small and flexible plastic needle, Flexi-Tip. The autoclavable Flexi-Tip in combination with the smooth consistency is time saving and helps for precise and deep application.

Composition: Calcium hydroxide >41% Barium sulphate Sterile isotonic saline solution pH 12.4

## Calasept<sup>®</sup> & Calasept<sup>®</sup> Plus

calasept Plus & Calasept are pure calcium hydroxides. The very high concentration of >41% in combination with optimal calcium ion release generates the extremely efficient and long-lasting antimicrobial effect. Calasept Plus & Calasept are radio-

Calasept is a ready-to-use paste in air tight syringe for direct application through the Paste Needle or Flexi-Tip.

The air tight syringes eliminate the risk of incorporating air into calcium hydroxide turning the material into calcium carbonate. Calcium carbonate is not possible to re-wet into calcium hydroxide.

### INDICATIONS

Pulp capping Pulp protection Insulation in deep cavities Stepwise excavation

**DID YOU KNOW?** Calasept has a bactericidal effect on carious dentine.

Calasept creates a defensive response promoting formation of hard tissue barriers.

Calasept stored in the refrigerator will not prolong shelf-life, only make the consistency thicker.





The radiograph of 35 with periapical osteitis.




Two cavities after pulpotomy of two small lesions, showing haemostasis in the mesial, first prepared cavity (arrow) and cessation of bleeding in the distal cavity.



Stepwise excavation

After gross excavation, the deepest layer of carious dentine will be covered with Calasept.

#### TEST: Determination of release of calcium ions by atomic absorption.

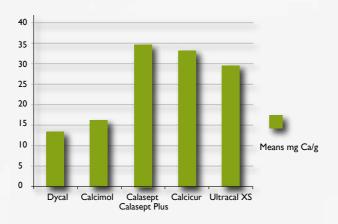

#### Conclusions

A significant difference in release of calcium ions was observed between setting and pure materials. Calasept showed the highest quantity of calcium release. The values were obtained after 7 days, crucial time when closing a pulp, the values can vary due to draining of molecule content in this study.

Dr. Prof. Paz Alejandro University J F Kennedy Argentina, University la Plata.

Argentina 2007/2008





3 months later the periapical radiolucency has been replaced with new bone promoted by the Calasept.



Appearance of hard-tissue barrier 3 months after pulpotomy.





After 6 weeks the cement was removed, Calasept washed away and the excavation of the carious dentin completed without pulp exposure.

### reasons to use **Calasept<sup>®</sup> Temp**

### **Rapid self curing** in the cavity

Calasept Temp is a temporary filling material consisting of zinc oxide, glycerine and calcium sulphate - hemihydrate. The material is rapidly self curing in the cavity having a high adhesion to dentin.

### High adhesion to dentin

Quickly sets in the cavity, providing a tight seal. High resistance to chewing pressure.

### **Outstanding** marginal seal

Calasept Temp guarantees an outstanding marginal seal due to its slight expansion.

### **Excellent** adaptability

The material is pre-mixed and is simple to apply and to remove. Excellent resistance to chewing pressure. No irritation to the pulpa nor the gingiva.





## **Calasept<sup>®</sup> Temp**

CALAS

Prod No: 1280000 Calasept Temp, 3 x 30 gram

reliable closing of the cavity.

Compositions: Zink oxide: 15% Glycerine: 20% Calcium sulphate – hemihydrate: 50%

### self curing paste for a reliable closing of the cavity

CALASEPT Ten

Calasept Temp is a temporary filling that is ready to use and requires no mixing. The material is rapidly self curing in the cavity and has a high adhesion to dentin. This together with the resistence against chewing-pressure gives a very