Dental Conservative Instruments-Crown and Matrix Handling

Matrix Retainers/Tensioners, Crown Removers, Crown Scissors & Forceps

ErgoDenta ApS

Hørskættø 14, 2630 Taastrup, Denmark.

Tel: +4555241100

Email: info@ergodenta.dk

Web: www.ergodenta.dk

en-English

Attention

The operator/user must carefully read and understand this manual thoroughly to keep the product performance durable and reliable for defined life expectancy. After opening the packaging, first of all it is necessary to check the component(s) against the standard configuration. Check that these are all present and in perfect condition.

Intended Use:

These instruments are intended to be used for the secure placement, stabilization, and removal of crowns and matrix bands during dental restorative procedures. They aid in the accurate adaptation of matrix bands, removal of crowns, and trimming of restorations, ensuring precision and efficiency.

The Instrument Specific Intended Use is given below:

Matrix Retainers/Tensioners:

Intended to be used for securing and tightening matrix bands around teeth to achieve proper adaptation, prevent material overhangs, and ensure stable restorations.

Crown Removers:

Intended to be used for applying a controlled force for the efficient removal of temporary or permanent crowns while preserving the integrity of the tooth structure.

Crown Scissors:

Intended to be used for cutting and trimming crowns, bridges, and restorative materials to achieve optimal fit and contour.

Crown Forceps:

Intended to be used for gripping and removing crowns, bridges, and similar restorations while ensuring atraumatic detachment.

 Our Dental Conservative Instruments are intended to be used by qualified dental professionals only in a professional dental facility.

Patient population:

These instruments are intended to be used on all patient populations. However, the instrument type/variant must be selected as per patient keeping in view the examination/treatment requirement.

For Available Configurations/Models please visit the Restoration section in our Catalogue

Features

Matrix Retainers/Tensioners:

- Adjustable tensioning mechanisms ensure precise adaptation of the matrix band, minimizing gaps and overhangs.
- Stainless steel construction ensures durability and resistance to corrosion.
- Locking systems prevent slippage during procedures, ensuring stable restorations.

Crown Removers:

- Designed to apply controlled force for efficient and atraumatic crown removal.
- Specialized, serrated, and grooved tips ensure a secure grip to minimize slippage.
- Available in straight, angled, and silicone-tipped designs to enhance accessibility and minimize restoration damage.

Crown Scissors & Forceps:

- Crown Scissors have Sharp-edged blades enabling precise cutting and shaping of crowns and bridges.
- Crown Forceps have Serrated or grooved tips ensuring a firm grip for atraumatic removal of crowns.
- Designed to distribute force evenly, minimizing the risk of deformation or fracture.

Operating instructions

 These instruments are non-sterile and must be cleaned and sterilized before first use.

 Always check if the reprocessed instruments are in perfect condition and do not use the instruments if any kind of deterioration, damage or deformation is observed.

 Use a firm but controlled grip to maintain precision and stability.

For Matrix Retainers/Tensioners:

- Select the appropriate matrix band and retainer type for the procedure.
- Securely tighten the matrix band around the tooth, ensuring proper adaptation.
- Ensure a stable fit to prevent movement during restoration placement.

For Crown Removers:

- Select the appropriate tip for the crown type and location.
- Apply controlled pressure to disengage the crown without excessive force.
- Utilize an appropriate technique to prevent damage to the underlying structure.

For Crown Scissors & Forceps:

- For scissors, use smooth, controlled cutting motions to trim restorations precisely.
- For forceps, ensure a secure grip on the crown before applying removal force.
- Avoid sudden force to prevent damage to the restoration or surrounding structures.

 In case of Crown Scissors & Forceps: Before use ensure that blades and gripping surfaces are free of debris, properly aligned, and fully functional. Check the cutting efficiency of scissors before each use to avoid excessive pressure and damage to the restoration

Adverse Event

Any serious incident that has occurred concerning the device should be reported to the manufacturer and to the respective competent authority of the Member State in which the incident has occurred.

Cleaning & Sterilization

Cleaning

 Clean the device with care after each application to avoid the drying of contaminants.

Initial Cleaning

1. Remove any visible debris from the instrument using moisten a cotton swab or cloth with isopropyl or ethyl alcohol and gently wiping the surface of the product.
2. Use a soft brush and enzymatic cleaner to remove contaminants thoroughly.

Manual Cleaning

3. Prepare a solution of neutral pH enzymatic detergent according to the manufacturer's instructions.
4. Immerse in a pre-soak enzymatic cleaner solution
5. Thoroughly clean all surfaces of the Instrument using a soft brush or cloth. Pay close attention to any crevices or hard-to-reach areas.
6. Rinse thoroughly under lukewarm running tap water for a minimum of 30 seconds to remove all traces of detergent.

Automated Cleaning

1. Always clean in an appropriate box or cassette and ensure that they are properly positioned within the unit.
2. Always follow the equipment manufacturer's instructions and ensure the instruments are compatible with the cleaning system.
3. Visually inspect for cleanliness, If any contamination is visible, repeat the cleaning steps until required cleanliness is achieved.
4. Dry with clean, lint free cloth or filtered compressed air until there is no visible moisture. Follow STEAM STERILIZATION PROCEDURE.

 Ultrasonic cleaning may be performed in accordance with hospital or facility protocols. Ensure proper rinsing and drying after the process to prevent residue buildup.

Sterilization

Note: Ensure that the instruments are completely dry before sterilization.

Steam Autoclaving

1. Place the instrument in an approved sterilization pouch or wrap.
2. Sterilize using a steam autoclave following a validated sterilization cycle according to ISO 17665-1 and your facility's protocols. A typical recommended cycle is 132-134°C (270-273°F) for 4 minutes.
3. Verify no residual moisture remains after sterilization to prevent corrosion and ensure longevity.
4. Ensure that the hinges are lubricated with an appropriate medical-grade lubricant after sterilization to maintain smooth operation.
5. Allow the instruments to cool and dry completely before handling. Store in a clean, dry, and controlled environment until ready for use.
6. Follow the pouch/wrap manufacturer instructions for storage conditions and maximum storage time.
7. For scissors and forceps, periodically check the hinge movement to ensure continued smooth function.

Important Notes

- Always Inspect the instruments for any signs of deterioration, breakage and bent etc. after each sterilization cycle. Discard if damage is observed.
- Always follow your facility's infection control protocols.
- Refer to your autoclave manufacturer's instructions for proper operation and maintenance. The Sterilization equipment must be validated by the hospital and or sterilization equipment manufacturers.
- The above-mentioned cleaning and sterilization guidelines, provided by manufacturer are intended as procedures compatible with specific materials. Sterilization must be performed according to the Hospital/Clinic approved protocol.
- The responsibility for ensuring proper sterilization of instruments lies with the user. The effectiveness of sterilization depends on validated cleaning, packaging, and sterilization procedures carried out at the facility.

Disposal

Our conservative instruments: Matrix Retainers/Tensioners, Crown Removers, Crown Scissors & Forceps contain no hazardous materials. However, these must be cleaned/sterilized as per defined procedures before disposition. Please follow the local and national regulations or healthcare facilities' defined disposal and waste management policies.

Disclaimer

The products must be used, reprocessed, and maintained strictly in accordance with the instructions provided above. Any deviation from these guidelines by the dental professional or user is undertaken at the user's sole risk. The Manufacturer will not accept any requests for refunds or exchanges under warranty for products that have not been handled and reprocessed in compliance with these instructions.

Explanation of utilized symbols

	Medical Device Compliant with Regulation (EU) 2017/745
	Consult instructions for use
	Medical Device
	Product Code/Catalogue Number
	Model/Variant
	Lot Number
	The Device is supplied non-sterile.
	Sterilizable in a steam sterilizer (autoclave) at 132-134°C (270-273°F)
	Manufacturing Date
	Manufacturer
	Keep in a cool, dry place
	Caution, read instructions

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