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EN	Porcelain etch
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY TO BE USED BY DENTIST ONLY	

COMPOSITION: hydrofluoric acid 9,5%, base gel, dye.

INTENDED USE
The product is used for porcelain etching when preparing prosthetic restorations, or for etching porcelain veneers, crowns or inlays and onlays before their cementation.

INSTRUCTIONS FOR USE
Before the treatment immerse attached applicator in alcohol (ethyl alcohol) and then fix it onto the syringe.
1. Isolate the treatment field with a rubber dam.
2. Screw the elastic blue applicator on the syringe with the etchant, replacing the as sty plus with this applicator.
3. Apply a small amount of etching preparation on the prepared and dried surface. Immediately close the syringe with the safety screw.
4. Leave the etching preparation on for 60 seconds and then rinse and dry the procedure area.
5. Apply Silane according to the instruction for use provided by the manufacturer. Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

CONTRAINDICATIONS
Do not use the product for patients with hypersensitivity to the product's components.

PRECAUTIONS
The preparation is corrosive: avoid contact with the skin and mucosa membranes. In case of contact, rinse immediately with plenty of running water and contact a doctor.

TRANSPORT
Avoid contact with eyes. In case of contact, rinse with plenty of water holding the eyelid wide open and seek ophthalmologist's advice. If swallowed, do not induce vomiting. Rinse mouth with water. Contact a doctor.
Use a rubber dam and protection for patient's eyes when repairing teeth in the patient's oral cavity. Use a vacuum system while etching. Equip staff with protective clothing, gloves and visors.
Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE
Store in a tightly closed original container below 25°C. Keep out of reach of children. The expiry date is on the direct packaging. Once container is open, the expiry date doesn't change if the container is securely closed after each use.

TRANSPORT
The manufacturer allows transport of the product for a period not longer than 10 days at a temperature not exceeding 40°C. Short-term transport within the given temperature range does not affect the safety and efficacy of the product throughout its shelf life.

HANDLING THE PRODUCT CONTAINER
Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING
Syringe of 2 ml of preparation.
Last update: 11.02.2025/11

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