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EN	Oil for dissolving gutta-percha
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY	
TO BE USED BY DENTIST ONLY	

COMPOSITION: eucalyptus oil 100%

INTENDED USE

EUCALYPTOL is intended for use during repeated endodontic treatments to remove gutta-percha from the root canal.

INSTRUCTIONS FOR USE

Dip the attached pipette in ethanol and allow to dry before the procedure.

- Using the pipette, insert 1-2 drops of liquid into the root canal.
- Remove gutta-percha.
- Use another dose of preparation if necessary.
- If gutta-percha is still dry or crumbly, apply EUCALYPTOL again.
- Repeat loop as needed.

The product is reusable while the pipette is disposable. Multiple use of the pipette may cause the risk of re-infection.

CONTRAINDICATIONS

Do not use for patients who are hypersensitive to the product components.

PRECAUTIONS

It can cause allergic skin reaction.

Gutta-percha dissolving oils can reduce dentin microhardness and adversely affect the interaction of dentin with tooth restoration materials. Flammable liquid and vapour. The EUCALYPTOL product has cytotoxic potential. Endodontic treatment may involve temporary postoperative pain. Use a cofferdam, disposable gloves, safety goggles/visors and protective clothing when handling the product. Keep out of the reach of children.

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Store in original packaging below 25°C. Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Bottle with 10 ml of preparation, application pipettes (x3).

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