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BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY TO BE USED BY DENTIST ONLY

COMPOSITION: 4-ethyl rhodamine, propylene glycol, surfactants.

INTENDED USE

RED DETECTOR is intended for locating caries lesions. Red dye in the product permeates the tissue with caries lesions and dyes it red. RED DETECTOR increases safety of work by locating the carious dentine and saving the healthy tissues.

INSTRUCTIONS FOR USE

Before the treatment, immerse the supplied applicator in ethyl alcohol, allow it to dry and then install the applicator on the syringe. After removing all visible carious lesions, apply one drop of preparation on the loss area, wait 5-10 seconds, then irrigate and dry the cavity. Points dyed bright red indicate carious dentine.

After using the product, withdraw the syringe piston to prevent overflow of preparation, screw off the applicator and then screw the white plug on the syringe.

Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

PRECAUTIONS

It irritates the eyes.
Use a cofferdam, disposable gloves, safety goggles/visors and protective clothing when handling the product. Keep out of reach of children.
Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Store in original packaging below 25°C. The expiry date is on the direct packaging.

Once container is open, the expiry date doesn't change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Syringe with 2 ml of preparation, disposable brush applicators.

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