Diverse Support Products

Sterilization Cassettes/Trays, Instrument Trays & Containers

Dental Napkin Holder

ErgoDenta ApS

Hørskættø 14, 2630 Taastrup, Denmark.

Tel: +4555241100

Email: info@ergodenta.dk

Web: www.ergodenta.dk

en-English

Attention

The operator/user must carefully read and understand this manual thoroughly to keep the product performance durable and reliable for defined life expectancy. After opening the packaging, first of all it is necessary to check the component(s) against the standard configuration. Check that these are all present and in perfect condition.

Intended Use:

These products are intended to be used for the secure organization, protection, and sterilization of dental instruments, as well as for maintaining proper hygiene and workflow efficiency during procedures.

The Product Specific Intended Use is given below:

Sterilization Cassettes & Trays:

Designed to securely hold and organize dental instruments during cleaning, disinfection, sterilization, and storage. They assist in maintaining proper instrument handling and infection control.

Instrument Trays, Bowls & Containers:

Used for holding, organizing, and transporting dental instruments, materials, and solutions during procedures. They assist in maintaining accessibility, sterility, and procedural efficiency

Dental Napkin Holder:

Used for securing dental napkins or bibs around the patient's neck during dental procedures, ensuring hygiene and patient comfort.

 These products are intended to be used by dental support staff, hygienists, assistants, and other personnel involved in instrument handling, sterilization, and clinical preparation. Users should be familiar with basic sterilization, infection control, and organizational procedures in a dental setting.

For Available Configurations/Models please visit the Diverse section in our Catalogue.

Features

Sterilization Cassettes & Trays:

- Securely hold dental instruments during sterilization, storage, and transport, preventing damage or contamination.
- Perforated design ensures even penetration of sterilizing agents for thorough disinfection.
- Organize instruments by procedure type, allowing quick and efficient access during treatments.
- Minimize the risk of cross-contamination between instruments and personnel.

Instrument Trays, Bowls & Containers:

- Provide a sterile and organized area for holding instruments and materials.
- Manufactured from high-quality, autoclavable stainless steel or medical-grade plastic for durability and infection control.
- Available in various sizes to accommodate different procedural requirements.
- Designed with smooth surfaces for easy cleaning and maintenance.

Dental Napkin Holder:

- Securely holds dental napkins or bibs in place, preventing displacement during procedures.
- Made of stainless steel for durability and patient comfort.
- The clips and chain provide a secure yet gentle hold.
- Easy to clean and disinfect, ensuring hygiene and patient safety.

Usage instructions

 Only authorized personnel should use these products following established clinical protocols. Avoid excessive force when handling to prevent bending or breakage. Do not use products that show signs of rust or corrosion.

Sterilization Cassettes & Trays:

- Open the cassette or tray and place cleaned dental instruments in designated slots.
- Ensure instruments do not overlap to allow proper exposure to sterilization agents.
- Close and securely lock the cassette before placing it in the sterilizer.
- Follow sterilization cycle guidelines as per institutional protocols.

Instrument Trays, Bowls & Containers

- Select the appropriate tray or bowl based on the procedural requirements.
- Arrange instruments and materials in an organized manner to ensure efficiency during use.
- After the procedure, clean and disinfect the tray/bowl before sterilization if required.

Dental Napkin Holder:

- Place the dental napkin or bib around the patient's neck.
- Secure the napkin holder clips and chain to keep it in place.

 Avoid excessive force when handling to prevent bending or breakage.

Cleaning & Sterilization

These products are classified as support items rather than medical devices; however, they require proper cleaning and sterilization to ensure hygiene and prevent cross-contamination. Care should be taken to prevent damage while maintaining sterility.

Cleaning

 Clean the Instrument trays, bowls and containers with care after each application to avoid the drying of contaminants.

Initial Cleaning

1. Rinse products under warm running water immediately after use to remove visible debris.

Manual Cleaning

2. Use a soft brush and neutral pH detergent to clean all surfaces, including slots and hinges.
3. Pay special attention to perforations in cassettes to ensure all residues are removed.
4. Rinse thoroughly and inspect for residual contamination before proceeding to sterilization.

Automated Cleaning

1. Always follow the equipment manufacturer's instructions and ensure that these support products are compatible with the cleaning system.
2. Ensure that the trays, cassettes and containers are positioned to allow water and detergent to reach all surfaces.
3. Visually inspect for cleanliness, If any contamination is visible, repeat the cleaning steps until required cleanliness is achieved.
4. Dry with clean, lint free cloth or filtered compressed air until there is no visible moisture. Follow STEAM STERILIZATION PROCEDURE.

 Ultrasonic cleaning may be performed in accordance with hospital or facility protocols. Ensure proper rinsing and drying after the process to prevent residue buildup.

Sterilization

Note: Ensure that the products are completely dry before sterilization.

Steam Autoclaving

1. Place the products in an approved sterilization pouch or wrap.
2. Sterilize using a steam autoclave following a validated sterilization cycle according to ISO 17665-1 and your facility's protocols. A typical recommended cycle is 132-134°C (270-273°F) for 4 minutes.
3. Do not stack any of these products tightly inside the sterilizer to allow proper steam circulation.
4. Verify no residual moisture remains after sterilization to prevent corrosion and ensure longevity.
5. Allow the instruments to cool and dry completely before handling. Store in a clean, dry, and controlled environment until ready for use.

6. Follow the pouch/wrap manufacturer instructions for storage conditions and maximum storage time.

Important Notes

- Always Inspect the instruments for any signs of deterioration, breakage and bent etc. after each sterilization cycle. Discard if damage is observed.
- Always follow your facility's infection control protocols.
- Refer to your autoclave manufacturer's instructions for proper operation and maintenance. The Sterilization equipment must be validated by the hospital and or sterilization equipment manufacturers.
- The above-mentioned cleaning and sterilization guidelines, provided by manufacturers, are intended as procedures compatible with specific materials. Sterilization must be performed according to the Hospital/Clinic approved protocol.
- The responsibility for ensuring proper sterilization of instruments lies with the user. The effectiveness of sterilization depends on validated cleaning, packaging, and sterilization procedures carried out at the facility.

Disposal

Our support products: Sterilization Cassettes/Trays, Instrument Trays & Containers and Dental Napkin Holders contain no hazardous materials. However, these must be cleaned/sterilized as per defined procedures before disposition. Please follow the local and national regulations or healthcare facilities' defined disposal and waste management policies.

Disclaimer

These products must be used, reprocessed, and maintained strictly in accordance with the instructions provided above. Any deviation from these guidelines by the dental professional or user is undertaken at the user's sole risk. The Manufacturer will not accept any requests for refunds or exchanges under warranty for products that have not been handled and reprocessed in compliance with these instructions.

Explanation of utilized symbols

	Medical Device Compliant with Regulation (EU) 2017/745
	Consult instructions for use

	Product Code/Catalogue Number
	Model/Variant
	Lot Number
	Sterilizable in a steam sterilizer (autoclave) at 132-134°C (270-273°F)
	Manufacturing Date
	Manufacturer
	Keep in a cool, dry place
	Caution, read instructions

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