Content

English	2	Bahasa Malay	110
Deutsch	8	Malti	116
Français	14	Nederlands	122
Italiano	20	Norsk	128
Español	26	Polski	134
Български	32	Português	140
Česky	38	Română	146
中文 (中华人民共和国)	44	Русский	152
Dansk	50	Slovenčina	160
Eesti	56	Slovenščina	166
ελληνική	62	Srpski	172
עברית	68	Suomi	178
Hrvatski	74	Svenska	184
íslenska	80	ภาษาไทย	190
한국어	86	Türkçe	198
Latviešu	92	Українською мовою	204
Lietuvių	98	Distributors	212
Magyar	104		

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Resorbable bilayer membrane for tissue regeneration

Composition

Geistlich Bio-Gide® / Geistlich Bio-Gide® Perio is a collagen membrane obtained by a standardised, controlled manufacturing process. The collagen is obtained from pigs certified by a veterinarian and is carefully purified to avoid immunologic reactions. Geistlich Bio-Gide® / Geistlich Bio-Gide® Perio has a bilayer structure: The porous surface, facing the bone, allows for the ingrowth of bone-forming cells. The dense surface, facing the soft tissue, prevents the ingrowth of fibrous tissue into the bone defect. The membrane consists of natural collagen without further cross-linking or chemical additives.

Geistlich Bio-Gide® / Geistlich Bio-Gide® Perio is sterilised in double-blister packs by γ -radiation. STERILE R

Properties / Effects

Low antigenicity and excellent biocompatibility favour the use of Geistlich Bio-Gide® / Geistlich Bio-Gide® Perio in surgery. Experimental data have shown that the duration of barrier function ensures sufficient bone formation time for the indications listed below. The fibrous microstructure of Geistlich Bio-Gide® / Geistlich Bio-Gide® Perio is hydrophilic. The membrane retains its structural integrity even when wet.

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Adaptation to the contours of the bone wall is easily achieved. Fixation with sutures or pins is possible.

Due to the collagen's natural origin, in the dry state a slight bulging and variations in the membrane thickness may be noted.

Geistlich Bio-Gide® Perio has a smooth, dense surface. As this prolongs the water uptake, cutting for periodontal application is facilitated.

Indications

Geistlich Bio-Gide® and Geistlich Bio-Gide® Perio are intended to be used as a barrier membrane in the regeneration of bone defects.

Geistlich Bio-Gide® is used after bone grafting procedures and to enhance soft tissue formation, e.g. in

- > GBR (Guided Bone Regeneration) procedures

- > Immediate or delayed augmentation around implants placed in extraction sockets
- > Reconstruction of the alveolar ridge
- > Filling of bone defects in the maxillofacial area
- > Cleft closure surgeries

Geistlich Bio-Gide® Perio is especially indicated for periodontal bone defects.

Because of its elasticity, Geistlich Bio-Gide®/Geistlich Bio-Gide® Perio should be used in combination with void-filling bone-graft materials such as autologous bone or bone substitutes (e.g. Geistlich Bio-Oss®) whenever possible.

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- > If necessary, the bone defect is surgically exposed and the usual surgical procedures (curettage etc.) are undertaken.
- > The defect is filled with a space-making material, such as autologous bone or bone substitute (e.g. Geistlich Bio-Oss®), whenever possible.
- > The membrane is trimmed to the required size with scissors. The use of a template (e.g. sterile aluminum foil) may be helpful. The Geistlich Bio-Gide® Perio package contains sterile stencil templates in 4 different basic periodontal shapes. The membrane should overlap the walls of the defect by at least 2 mm to allow complete bone contact and to prevent connective tissue ingrowth lateral to the material.
- > The dense surface (Geistlich Bio-Gide®: marked with «UP» at the membrane margin) should face the soft tissue, and the rough side should face the defect.
- > The membrane is applied over the defect and held in place with moderate pressure. The period of time necessary for the application of pressure will vary with the degree of bleeding. Adhesion to the bone surface is facilitated by the hydrophilic properties of the membrane.
- > Complete penetration of the membrane by blood and exudate allows perfect adaptation and adhesion of the membrane to the defect and facilitates the formation of a blood clot beneath the membrane.
- > Fixation of the membrane is possible due to its high tensile strength. Fixation may be indicated to avoid membrane displacement due to shear loading or mobilisation.
- > The soft tissue flap is sutured over the membrane securely and without tension (e.g. single sutures and deep mattress suture), if possible.

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- > In general, the wound should be closed completely to avoid possible accelerated absorption of the exposed membrane, if possible. Clinical experience demonstrates satisfactory healing in most cases when a complete wound closure is not possible.

Special instructions for use in periodontology

- > A basic requirement for successful periodontal treatment includes control of any bacterial infection as well as thorough oral hygiene. It is therefore advised that, preceding the surgical intervention, there be a hygiene phase, which would include proper instruction for the patient. A postoperative maintenance phase can ensure long-term therapeutic success.
- > In order to avoid the formation of a long junctional epithelium, Geistlich Bio-Gide® Perio must be adapted closely to the tooth (if necessary with additional fixation using suture material).

Limitation on use / Precautions

Contraindications

- > Geistlich Bio-Gide® / Geistlich Bio-Gide® Perio should not be used in the presence of infected wounds.
- > Geistlich Bio-Gide® / Geistlich Bio-Gide® Perio should not be used in patients with a known allergy to collagen.

Precautions

- Geistlich Bio-Gide® / Geistlich Bio-Gide® Perio should only be used by trained dentists and surgeons.
- Geistlich Bio-Gide® / Geistlich Bio-Gide® Perio should be used with special caution in patients with
 - > acute or chronic infection (e.g. osteomyelitis) at the surgical site
 - > uncontrolled metabolic diseases, e.g. diabetes, osteomalacia, thyroid disorder
 - > prolonged corticosteroid therapy
 - > autoimmune diseases
 - > radiotherapy
 - > heavy smoking

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Due to the adherence to the bone tissue and the elasticity of Geistlich Bio-Gide® / Geistlich Bio-Gide® Perio, the use of bone augmentation material is recommended to create and maintain space for bone regeneration. In the case of membrane exposure (e.g. soft tissue dehiscency) during the healing phase the resorption time may be accelerated.

Geistlich Bio-Gide® Perio: Do not implant the stencil templates! Their perforations enable them to be distinguished from the membrane.

Side effects

As the membrane is a collagen product, in very rare cases allergic reactions may occur. Due to the special physical properties and the prolonged absorption time an inflammatory reaction may occur in very rare cases. Possible complications which may occur with any surgery include swelling at

the surgical site, flap sloughing, bleeding, local inflammation, bone loss, infection, or pain.

Pregnancy / Lactation

Data on the use of the product during pregnancy or lactation are not available. For safety reasons: women who are pregnant or breastfeeding should not be treated with Geistlich Bio-Gide® / Geistlich Bio-Gide® Perio.

Other information

The content of the double-blister pack is intended for single use.

Re-sterilisation is not possible. Discard any unused material. Sterility and function are no longer guaranteed in the event of reuse.

Do not use the product after the indicated expiry date.

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Store in a dry place and at room temperature (15–25 °C).

The product is sterile, if the packaging is unopened and undamaged. If the sterile packaging has been damaged, the product must not be used.

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Bahrain

Gulf Pharmacy & General
Store
a division of Gulf
Corporation of Technology
P.O. Box 2576, Villa 390
New Zinj 330
Manama
Kingdom of Bahrain

Yousuf Mahmood Husain
W.L.L.
P.O. BOX 23
Bldg. 97, Block 328
Road 322
Salmaniya Ave.
Manama
Kingdom of Bahrain

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Estonia

Sirowa Tallinn AS
 Salve 2C
 111612 Tallinn
 Estonia

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Hong Kong

HEAT Consultancy Co. Ltd.
 1912-13, The Centre
 99 Queen's Road Central
 Hong Kong

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E. Bridde ehf
 Holtasmári 1
 201 Kópavogur
 Iceland

India

LifeCare Devices Private
 Limited
 LifeCare House
 841, Mori Road
 Mahim West
 Mumbai – 400016
 India

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Dent-Med Materials B.V.
Dorpsstraat 72
1713 HK Obdam
Nederland

New Zealand

Henry Schein Shalfoon
23 William Pickering Drive
Albany, Auckland 0632
New Zealand

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Philippines

Spectrumed Inc.
Unit 2001-B West Tower
Philippine Stock Exchange
Center
Exchange Road, Ortigas
Center
Pasig City 1605
Philippines

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Singapore

Raydent Supplies (S) Pte Ltd
6 Ubi Road 1 #04-01
Wintech Centre
Singapore 408726
Singapore

Slovakia

Sanitaria spol. s.r.o.
Sabinovská 4/A
821 02 Bratislava
Slovenská

Slovenia

Aurora Dental d.o.o.
Brilejeva Ulica 2
1000 Ljubljana
Slovenija

South Africa

Implants for Africa (Pty) Ltd.
Unit 30B
Platinum Junction Business
Park
School Street
P.O. Box 1028
ZA-7441 Milnerton

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Vietnam

D.O.E.
 (Dentistry Of Excellence)
 199 Nguyen Thi Minh Khai
 Dist.1 – Ho Chi Minh City
 Vietnam

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