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Sectional Matrix Bands: Matrix bands are single use disposable items. Sterilization prior to use is recommended and can be performed using any common sterilization equipment or chemicals.

Important Cleaning and Sterilization Instructions: Composi-Tight 3D™ Soft Face™ rings (orange tip) and thin tine G-Rings® (gray) will maintain their strength and appearance by using the following cleaning and sterilization procedure; **Steam autoclave sterilization only.** DO NOT use chemical vapor sterilizers, alcohol, acetone, ketone, or other harsh chemical disinfectants including ultrasonic cleaning solutions that contain these chemicals.

Rings: The orange Soft Face™ 3D-Ring will be the ring used in most circumstances because of its ease of placement and its ability to adapt to a wide variety of tooth anatomies while reducing flash and restoring proper contour. The gray thin tine G-Ring®, with its burnished fine ends, is utilized in cases where the shape of the dentition makes ring retention more problematic, such as between a canine and bicuspid, or on very short or malpositioned teeth.

Slick Bands #	Regular Bands #	Most commonly used for:
	M050	primary molars
	M100	bicuspid
	M150	bicuspid with deep cervical preparations
	M175	bicuspid and small molars
	M200	molars
	M300	molars with deep cervical preparations

Sectional Matrix Bands: Select based on tooth height and restoration type

Garrison
Dental Solutions

Carlstrasse 50
52531 UEbach-Palenberg
phone +49 2451 971 409
fax +49 2451 971 410
email info@garrisonsdental.net
web www.garrisonsdental.net

COMPOSI-TIGHT®
3D™
SECTIONAL MATRIX SYSTEM

User instructions.

The **Composi-Tight 3D™ Sectional Matrix System** is designed to create tight, anatomical contacts on Class II composite restorations. The contoured sectional matrix bands create proper tooth anatomy while two styles of rings create the required tooth separation and band adaptation for a tight, natural contact.

Patent pending, Non-sterile, disposable (matrix bands), RX only

Technique for Class II Restorations Utilizing the 3D Soft Face™ Ring

1. **Place rubber dam.** Isolation of the area helps to prevent contamination from blood and saliva. As with many dental procedures, aspiration of small parts is a valid concern.
2. **Prepare tooth with conservative Class II preparation.**
3. **Place contoured sectional matrix band.** Select a matrix band that most closely approximates the occlusogingival height of the tooth. Over-approximate tooth curvature buccolingually by rolling the contoured band with your fingers. This will help keep the matrix band ends from interfering with the Soft Face™ 3D-Ring placement. The band should be oriented with the concave edge towards the occlusal margin of the tooth. (figure 1)
4. **Insert wedge.** For optimal contour, gingival seal and tooth separation, use Garrison's Wedge Wands™ or G-Wedge™. Keep a finger on the matrix band to prevent the wedge from dislodging the band during insertion. Wedge firmly, a high viscosity resin packed into the preparation may displace the band away from the tooth. (figure 2)
5. **Apply the Soft Face™ 3D-Ring retainer (#3D500).** Spread the Soft Face™ 3D-Ring with the Composi-Tight Ring Placement Forceps* and place over the wedge (figure 3). NOTE: For secure gripping of ring tips to surface, dry teeth thoroughly before placement.
6. **Burnish the band.** Very Important: Burnish the band in the desired contact area against the adjacent tooth and make sure there is no spring-back of the band. This will ensure an excellent contact.
7. **Use your preferred etching, disinfecting and bonding technique following manufacturer's instructions. Apply your preferred composite resin following manufacturer's instructions.**
8. **Remove the Ring, wedge and band.** Because of the superior contact achieved with Composi-Tight 3D™, removal of the band may require a Composi-Tight Matrix Forceps (model #BFA-1) or hemostat. This is especially true with particular bonding agents that preferentially bond to metal. For easier removal choose Slick Bands™.
9. **Contour and polish the restoration.**

figure 1

figure 2

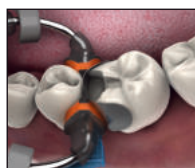
figure 3

figure 4

Stacking of Rings & MOD Restorations

Both the Composi-Tight 3D Soft-Face™ Ring (orange tip) and the thin tine™ G-Ring® (gray) are angled at 12.5 degrees to allow stacking of rings for performing M.O.D.s and multiple-tooth restorations. Either ring may be stacked over the other, and in any combination. (figure 4)

Technique for Class II Restorations Utilizing the thin tine G-Ring® (gray)

The thin tine G-Ring® (gray) is utilized in certain conditions where the shape of the dentition will not allow the contoured shape of the Soft Face™ 3D-Ring (orange tip) to grip tightly around the tooth, such as the distal of a canine. In this condition, the burnished ends of the thin tine G-Ring® (gray) will adapt and retain more effectively. The thin tine G-Ring® (gray) may also be preferred for malpositioned or pedodontic applications.

Optimal placement of the ring tines is between the wedge and band, but they may also be placed behind or on top of the wedge when conditions dictate (figure 5a-c).

figure 5a

figure 5b

figure 5c

* (Garrison part number AUMRDF-100 Ring Placement Forceps will hold the rings more securely and provide ample force to open the rings. NOTE: Most rubber dam clamp forceps will not open the 3D-Rings far enough to allow placement.)

Garrison
Dental Solutions

Carlstrasse 50
52531 Uebach-Palenberg
phone +49 2451 971 409
fax +49 2451 971 410
email info@garrisonsdental.net
web www.garrisonsdental.net

COMPOSI-TIGHT®
3D™
SECTIONAL MATRIX SYSTEM

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3D500

3D400

Important Cleaning and Sterilization Instructions: Composi-Tight 3D[™] Soft Face[™] rings (orange tip) and thin line G-Rings[®] (gray) will maintain their strength and appearance by using the following cleaning and sterilization procedure: **Steam autoclave sterilization only.** DO NOT use chemical vapor sterilizers, alcohol, acetone, ketone, or other harsh chemical disinfectants including ultrasonic cleaning solutions that contain these chemicals.

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figure 5b

figure 5c

*(Garrison part number AUMRDF-100 Ring Placement Forceps will hold the rings more securely and provide ample force to open the rings.

NOTE: Most rubber dam clamp forceps will not open the 3D-Rings far enough to allow placement.)

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Important Cleaning and Sterilization Instructions: Composi-Tight 3D™ Soft Face™ rings (orange tip) and thin line G-Rings® (gray) will maintain their strength and appearance by using the following cleaning and sterilization procedure: **Steam autoclave sterilization only.** DO NOT use chemical vapor sterilizers, alcohol, acetone, ketone, or other harsh chemical disinfectants including ultrasonic cleaning solutions that contain these chemicals.

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COMPOSI-TIGHT®
3D™
SECTIONAL MATRIX SYSTEM™

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3D400

Important Cleaning and Sterilization Instructions: Composi-Tight 3D™ Soft Face™ rings (clear tip) and

thin G-Rings® (gray) will maintain their strength and appearance by using the following cleaning and sterilization procedure: **Steam autoclave sterilization only.** DO NOT use chemical vapor sterilizers, alcohol, acetone, ketone, or other harsh chemical disinfectants including ultrasonic cleaning solutions that contain these chemicals.

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