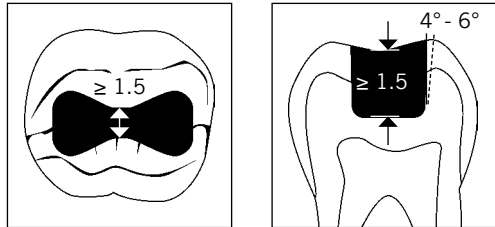
VOCO

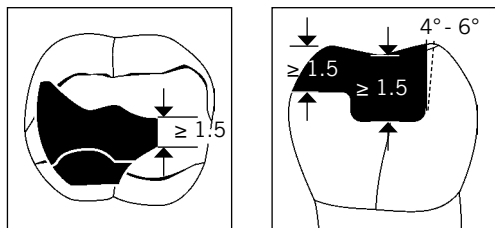
Grandio® disc


1) Inlay


2) Onlay

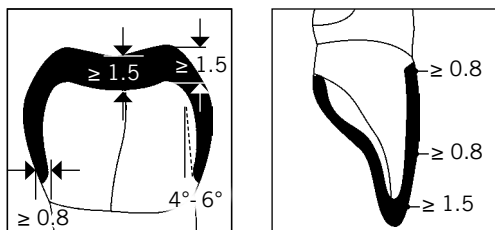3) Crown

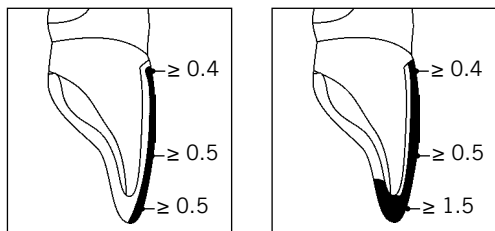4) Veneer

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MD EU Medical device

Product description:

Grandio disc is a nano-hybrid composite CAD/CAM disc with 86% by weight of inorganic fillers for the manufacture of customised, permanent single-tooth restorations and/or framework-supported prosthetic restorations.

Intended purpose:

Customised crowns, inlays, onlays, veneers and implant-retained crowns as well as veneers of frameworks of indirect restorations in the anterior and posterior regions.

Indication:

Lack of tooth substance, partial edentulism and complete edentulism.

Fields of application:

Without framework

- Inlays, onlays, veneers
- Crowns, implant-retained crowns

With supporting framework

- Veneers of supporting frameworks irrespective of the selected manufacturing process.

Contraindications:

Grandio disc should not be used in patients with known hypersensitivity (allergy) to its substances.

Patient target group:

Grandio disc is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Grandio disc should only be used by a professionally trained dental practitioner.

Use:

Shade selection:

Determine the shade on the cleaned tooth, before anaesthesia and application of a rubber dam, using a VITA® classical shade guide. Do this with natural daylight or room lighting as the light source. Operating theatre lights are unsuitable for shade selection, as their colour temperature is different from daylight.

Preparation

The preparation method to be selected depends on the specific application. Undercuts should be avoided irrespective of the type of application.

Without framework

As a general rule, the core/cavity preparation should be performed in accordance with the conventional rules for all-ceramic restorations:

- round off internal corners and edges
 - shoulder preparation with rounded-off internal edges or chamfer preparation
- Prior to fitting, any residues left by a temporary restoration and other contaminants must be removed mechanically and the area then cleaned with water. The restoration made of **Grandio disc** can then be inserted for checking purposes.

With supporting framework

Select core/cavity preparation methods appropriate for the framework material being used. Please observe and follow the respective manufacturer's specifications.

Design parameters

The design parameters to be selected depend on the specific application.

Design parameters without framework

Cement space:

70 µm (± 10 µm)

Crowns, inlays and onlays (see Fig. 1-3):

- preparation angle of 4°-6°
- minimum wall thickness: occlusal 1.5 mm – buccal 0.8 mm – cervical 0.8 mm
- minimum wall thickness below load-bearing cusp 1.5 mm

Veneers (see Fig. 4):

- minimum wall thickness: cervical 0.4 mm – labial 0.5 mm – incisal 0.5 mm
- minimum material thickness for covering incisal edge: 1.5 mm

Design parameters with supporting framework

Bonding space

For the bonding, a sufficiently large gap must be maintained between the components to ensure targeted processing. The structural design and specifications of the materials used in the bond must also be taken into consideration.

Design

The shape of the framework should be designed such that it follows the natural anatomical shape and provides homogeneous support. The framework must be designed in such a way that the restoration made of **Grandio disc** is supported at all points. Compliance with the specifications for the minimum wall thicknesses for crowns must be ensured (see fig. 3).

Grinding or milling process

Please pay attention to the specifications of the respective CAD/CAM systems. Should your CAD/CAM system not have a strategy for **Grandio disc** and/or **Grandio disc multicolor**, please consult your CAD/CAM system provider to arrange the installation of corresponding strategies. Note that every system manufacturer recommends appropriate tools for proper processing.

CAUTION: When using **Grandio disc multicolor**, please ensure that the colour orientation matches the digital orientation.

Completion

Finishing and polishing

The finishing and polishing of the restorations made of **Grandio disc** can be performed either intraorally or extraorally with the conventional tools and established techniques for composite processing.

Modification – staining and/or facing work

Roughen the surface of the restoration (for example with a diamond bur) and apply a suitable adhesive system (e.g. **Futurabond U**) in accordance with the instructions for use. Then apply the composite/ORMOCER® for characterisation (e.g. **GrandioSO**, **GrandioSO Flow**, **Admira Fusion Flow**) in accordance with the respective instructions for use.

Preparation for luting / bonding

Conditioning of the restorations made of Grandio disc

For an optimal bond, blast the luting surface of the restoration made of **Grandio disc** with aluminium oxide (25–50 µm) at 1.5–2 bar.

Note: Do not etch with hydrofluoric acid or phosphoric acid.

Clean the pretreated restoration using an ultrasonic bath or steam cleaner and then dry with oil-free air. Final cleaning with medical alcohol is possible.

Treat the luting surface with a silane coupling agent such as **Ceramic Bond**. The instructions for use must be observed and followed.

Conditioning of the framework

Prior to bonding, ensure that the restoration made of **Grandio disc** can be positioned on the framework in accordance with the plan.

In the further course of work, please observe and follow the specific instructions for use for conditioning the framework, even when using an opaque.

Luting / bonding

Intraoral luting to dental hard tissue

The restoration produced must be luted with a composite-based, adhesive luting agent. The instructions for use must be observed and followed. **VOCO** recommends the dual-curing luting system **Bifix QM** in combination with **Ceramic Bond** and **Futurabond U** for an optimal bond to the restoration.

Alternatively, it is also possible to use a purely light-cured, flowable composite/ORMOCER® (e.g., **GrandioSO Flow**, **Admira Fusion Flow**, **Bifix Veneer LC**) if the restoration has thin walls (≤ 2 mm). The instructions for use must be observed and followed.

Extraoral luting to framework

The restoration produced must be luted with a composite-based, adhesive luting agent. The instructions for use must be observed and followed.

Warnings, precautionary measures:

- Employ suitable suction measures during the work steps as applicable. Do not inhale dust produced during grinding.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content):

Barium aluminium borosilicate glass, silicon dioxide, polymethacrylate, stabilisers, pigments

Storage:

Storage at 4°C-28°C. Do not store the product in a place where it is exposed to direct artificial light or sunlight. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Grandio disc** must be reported to VOCO GmbH and the responsible authority.

Note:

The Summary of Safety and Clinical Performance of **Grandio disc** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at www.voco.dental.

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VC 60 AA6051 E1 0625 99 © by VOCO
ORMOCER® is a registered trademark of Fraunhofer Gesellschaft zur Förderung der angewandten Forschung e.V., München, Germany.
VITA® is a registered trademark of VITA Zahnfabrik H. Rauter GmbH & Co.