AH Plus[®] AH Plus Jet[®]

Next Generation

Endodontic Sealer
Scellant endodontique
Sigillante endodontico
Wurzelkanal-Sealer
Sellador endodóntico
Selante endodôntico
Endodontische sealer
Endodontisk sealer
Endodontisk forseglar
Endodontisk forseglingsmiddel
Endodonttinen täyte
Endodontyczna pasta uszczelniająca
Endodontinis sandariklis
Endodontiskais aizpildītājs
Endodontiline juuretäidis
Endodontická výplň
Endodontický výplňový materiál
Endodonciás tömőanyag
Sigilant endodontic
Endodontsko polnilo

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AH Plus® AH Plus Jet® Endodontic Sealer

Next Generation

CAUTION: This is a medical device. For dental use by dental professionals only.

USA: Rx only.

Content	Page
1 Product description	3
2 Safety notes	4
3 Step-by-step instructions	6
4 Hygiene	8
5 Lot number, expiration date and correspondence	9

1 Product description

AH Plus® endodontic sealer is a two-component paste-paste permanent root canal sealer based on epoxy-amine resin. It is available in two delivery forms: **AH Plus®** endodontic sealer in tubes, for an easy manual mix, or as **AH Plus Jet®** endodontic sealer double-barrel syringe with disposable mixing tips for intraoral use, offering a more precise, convenient and faster procedure. It functions as part of the endodontic treatment, allowing for restoration of tooth integrity. It adapts closely to the walls of the prepared root canal and presents permanent sealing ability and radiopacity. **AH Plus®** endodontic sealer does not stain teeth.

1.1 Intended Purpose

Permanent root canal sealer

1.2 Indications

- Permanent obturation of root canals of teeth of the secondary dentition in combination with root canal points.

1.3 Contraindications

- Use in patients with a history of allergic reaction to epoxy resins or amines or any of the other components (see [1.5 Composition](#) and [2.1 Warnings](#) for further details).

1.4 Delivery forms

- AH Plus® endodontic sealer in tubes for manual mixing of paste 1 and paste 2.
- AH Plus Jet® endodontic sealer double-barrel syringe with disposable mixing tips for intraoral use, offering a more precise, convenient and faster procedure.

1.5 Composition

Paste 1 (amber color):

- Calcium tungstate
- Araldite GY 250 epoxy resin
- Araldite GY 285 epoxy resin
- Highly dispersed silicon dioxide
- Polymeric additive
- Iron oxide pigments

Paste 2 (white color):

- Calcium tungstate
- Dibenzyl diamine
- Aminotetralin
- Highly dispersed silicon dioxide
- Cyclohexane diamine
- Polydimethylsiloxane
- Polymeric additive

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

Information symbol.

This information symbol is used to highlight important information for the correct use.

2.1 Warnings

The material contains epoxy resins Araldite GY 250 (Bisphenol A epoxy resin; 4,4'-isopropylidenediphenol, oligomeric reaction products with 1-chloro-2,3-epoxypropane; CAS Nos. 25068-38-6 and 1675-54-3) and Araldite GY 285 (Bisphenol F epoxy resin; formaldehyde, oligomeric reaction products with 1-chloro-2,3-epoxypropane and phenol; CAS No. 9003-36-5) and Cyclohexane diamine (3-aminomethyl-3,5,5-trimethylcyclohexylamine; IPDA; CAS No. 2855-13-2), which may be irritating to skin, eyes, and oral mucosa and may cause allergic contact dermatitis, allergic sensitization, or systemic allergic reactions in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.
- **Do not re-use contaminated tubes or syringes** to avoid cross-contamination.
- **Do not use improperly mixed material**, as wrong mix ratio or inhomogeneous mixing may lead to inadequate curing of the material and leaching of monomers or oligomers.
- **Prevent ingestion.** If accidental swallowing occurs, have patient drink plenty of water. If nausea or illness develops, seek medical attention.
- **Do not place too much sealer** to prevent overfilling of the root canal, damage of periapical tissue, acute inflammation of periapical tissues resulting in (temporary) post-operative pain or extrusion into adjacent anatomical structures, like the mandibular canal or the maxillary sinus.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the root canal filling. Use adequate isolation such as rubber dam.
- The tubes cannot be reprocessed. To prevent from exposure to spatter or spray of body fluids or contaminated hands, it is mandatory that the tubes are handled offside the dental unit with clean/disinfected gloves. Discard tubes if contaminated.
- The syringe cannot be reprocessed. To prevent from exposure to spatter or spray of body fluids or contaminated hands, the syringe should be handled offside the dental unit with clean/disinfected gloves. For intraoral use, apply Disposa-Shield® barrier sleeve over AH Plus Jet® endodontic sealer syringe before use with each patient. Discard syringe if contaminated.
- AH Plus Jet® mixing tip and Disposa-Shield® disposable barrier sleeve are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination or malfunction.
- Make sure that AH Plus Jet® mixing tip is correctly and firmly attached to the syringe and the intraoral tip to the mixing tip.
- Interactions:
 - If refrigerated, allow material to reach room temperature prior to use.
 - Do not use eugenol-containing materials in conjunction with this product, since they may interfere with setting of the product.
 - Tooth surfaces to be treated with dental adhesives should be free of sealer.

2.3 Adverse reactions

Following adverse reactions or undesirable side-effects might occur. See [2.1 Warnings](#) and referenced chapters for details on how to prevent or react.

- Acute inflammation of periapical tissues resulting in (temporary) post-operative pain (see [3.3 Application](#))
- Irritation and possible corneal damage, if material comes into contact with eyes
- Irritation of skin or possible allergic contact dermatitis, if material comes into contact with skin
- Inflammation of mucous membranes, if material comes into contact with mucous membranes
- Nausea or illness, if ingested
- Dys- or anaesthesia could occur if material is overfilled and extruded into the mandibular canal (see [2.1 Warnings](#))
- Ingestion or aspiration of mixing or intraoral tip (see [2.2 Precautions](#) and [3.2.2 AH Plus® Jet endodontic sealer \(syringe\)](#))

In general, root canal treatment and obturation can be associated with sinus discharge, intermittent dull ache, and intermittent discomfort on biting and, therefore, could occur with the use of this material as well.

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store at temperatures between 2 °C and 24 °C (36 °F and 75 °F). Use the product at room temperature.
- Do not use after expiration date.

3 Step-by-step instructions

3.1 Preparation

1. Prior to the application of the material, prepare, clean, and dry the root canals to be filled using state-of-the-art endodontic techniques.

3.2 Dosage and mixing

3.2.1 AH Plus® endodontic sealer (tubes)

Cross-contamination - To reduce risk of infection

1. The tubes cannot be reprocessed.
2. To prevent from exposure to spatter or spray of body fluids or contaminated hands, it is mandatory that the tubes are handled offside the dental unit with clean/disinfected gloves. Discard tubes if contaminated.
3. Dispose of contaminated tubes in accordance with local regulations.

Appropriate paste condition

Slight separation can occur to both pastes. This condition does not negatively affect the setting time of the mixed product. If the dispensed material is solely of clear color, discard the dispensed material, to ensure convenient handling of the mixed paste.

Intermixing of pastes

Do not use pastes 1 or 2 of AH Plus® Endodontic Sealer with pastes A or B of AH Plus® Root Canal Sealer.

1. Using a metal spatula¹, mix equal volume units (1:1) of paste 1 (amber color) and paste 2 (white color) of AH Plus® endodontic sealer on a glass slab or the mixing pad supplied with the package. Mix to a homogeneous consistency.
2. Tightly close tubes immediately after use.
3. Do not exchange caps of tubes. The white cap belongs to paste 1; the colored cap belongs to paste 2.

3.2.2 AH Plus® Jet endodontic sealer (syringe)

Cross-contamination - To reduce risk of infection

1. The syringe cannot be reprocessed.
2. To prevent from exposure to spatter or spray of body fluids or contaminated hands, the syringe should be handled offside the dental unit with clean/disinfected gloves.
3. For intraoral use, apply Disposa-Shield® disposable barrier sleeve over AH Plus Jet® endodontic sealer syringe before use with each patient.
4. Discard syringe if contaminated.
5. Dispose of contaminated syringes in accordance with local regulations.

Appropriate mixing ratios

The syringe filling volumes of pastes 1 (amber color) and 2 (white color) vary slightly. To ensure appropriate mixing, bleed the syringe prior to the first use.

¹ Mixing ratio by weight is 1 g of paste 1 to 1.26 g of paste 2.

1. Remove the cap by turning it 90° counter-clockwise and pulling it. For future use, set it aside to prevent contamination.
2. Attach the mixing tip ² to the syringe, aligning the notch.
3. Turn the tip 90° clockwise and secure the mixing tip in place.
4. Rotate and adjust the tip in angle in order to meet treatment requirements.
5. Carefully apply steady pressure onto the plunger. The two pastes are automatically mixed in equal volume units.
6. The sealer can be applied directly into the canal or extruded conventionally on a mixing pad.
7. For intraoral use, apply Disposa-Shield® disposable barrier sleeve over AH Plus Jet® endodontic sealer syringe before use with each patient.
8. After having finished, remove and discard mixing tip (together with protective barrier, if any) by turning it 90° counterclockwise and pulling it off.
9. Using clean/disinfected gloves, replace the cap on the syringe head, aligning the notch, then turn it 90° clockwise.

3.3 Application

Overfilling - To reduce the risk of damage of periapical tissues and acute inflammation of periapical tissues resulting in (temporary) post-operative pain

1. Do not place too much sealer to prevent overfilling of the root canal. For condensation and warm gutta-percha obturation techniques apply only a light coating of sealer onto the canal walls.
2. If overfilling occurs, the material is usually tolerated very well by the surrounding tissue. If, however, material is pressed into the mandibular canal or the maxillary sinus immediately apply state of care measures.

Clogging - To reduce the risk of separation of mixing tip from syringe

1. Do not allow material to set inside the mixing tip.
2. Do not reuse mixing tip.
3. Always make sure mixing tip is properly connected to syringe before application. The material should extrude easily.

3.3.1 Condensation and warm gutta-percha techniques

1. As a standard technique, the material is used in combination with gutta-percha.
2. For these filling techniques, where most of the canal is obturated by endodontic point material, apply a light coating of sealer with a paper point onto the canal walls to the working length and/or apply a light coating of sealer onto the surface of the root canal filling point(s).
3. Insert root canal filling point(s) into the canal according to standard procedure.
4. After obturation, remove all excess sealer from the access cavity.
5. Seal access cavity with appropriate restorative material.

3.3.2 Master-Point-Technique

1. Select a gutta-percha point (or alternatively a paper point) of the size of the last instrument used during apical preparation.
2. Wet the canal walls with the sealer through a pumping movement of the point. Alternatively, apply the sealer onto the tip of a Lentulo spiral.
3. Advance the Lentulo spiral slowly to the apex running at very low speed. Avoid the formation of air bubbles in the material and overfilling of the canal.
4. Withdraw Lentulo very slowly still running at low speed.

² AH Plus Jet® Mixing Tips, EU REF 60620116, US REF 667006

5. Dip disinfected and dry master point into the material and insert it into the canal with a slow pumping motion.
6. After obturation, remove all excess sealer from the access cavity.
7. Seal access cavity with appropriate restorative material.

3.3.3 Working and setting time

- Working time is 4 hours at 23 °C³.
- Setting time is 24 hours at 37 °C⁴.

3.4 Removal of root canal filling

1. If the material is used in combination with (gutta-percha) points, root canal fillings can best be removed using thermo-mechanical techniques.

4 Hygiene

Cross-contamination - To reduce risk of infection

1. Do not reuse single use products. Dispose of in accordance with local regulations.

4.1 AH Plus® endodontic sealer (tubes)

Cross-contamination - To reduce risk of infection

1. The tubes cannot be reprocessed.
2. To prevent from exposure to spatter or spray of body fluids or contaminated hands, it is mandatory that the tubes are handled offside the dental unit with clean/disinfected gloves. Discard tubes if contaminated.
3. Dispose of contaminated tubes in accordance with local regulations.

4.2 AH Plus Jet® endodontic sealer (syringe)

Cross-contamination - To reduce risk of infection

1. The syringe cannot be reprocessed.
2. To prevent from exposure to spatter or spray of body fluids or contaminated hands, the syringe should be handled offside the dental unit with clean/disinfected gloves. Do not reuse syringe if contaminated.
3. For intraoral use, apply Disposa-Shield® disposable barrier sleeve over AH Plus Jet® endodontic sealer syringe before use with each patient.
4. Discard syringe if contaminated.
5. Dispose of contaminated syringes in accordance with local regulations.

4.3 AH Plus Jet® mixing tip and Disposa-Shield® disposable barrier sleeve

The use of protective barriers is an additional precautionary measure against gross contamination but not against all contamination.

1. Right after use, carefully remove protective barrier together with mixing tip without contaminating the device.

³ Measured according to ISO 6876:2012

⁴ Measured according to ISO 6876:2012. The complete polymerization needs 7 days at 37 °C.

Cross-contamination - To reduce the risk of infection

1. Disposa-Shield® disposable barrier sleeve and AH Plus Jet® mixing tips are intended for single use only. Do not reuse in order to prevent cross-contamination or malfunction.
2. Discard in accordance with local regulations.

4.4 Cleaning

1. Clean spatulas, mixing slabs and instruments immediately after use with alcohol or acetone.

4.5 Disposal

Dispose of in accordance with local regulations.

5 Lot number, expiration date and correspondence

1. Do not use after expiration date. ISO standard is used: “YYYY-MM” or “YYYY-MM-DD”
2. The following numbers should be quoted in all correspondence:
 - Reference number (REF)
 - Lot number (LOT)
 - Expiration date (EXP)
3. Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.
4. Device Identification (Basic UDI-DI) : ++D010EFM02PZ

A summary of the safety and clinical performance (SSCP) for this product can be found (upon activation) at <https://ec.europa.eu/tools/eudamed> by searching using the Basic UDI-DI number listed above and at <https://www.dentsplysirona.com/ifu> using the reference number (REF).

For an explanation of the symbols used on the label, please refer to the Symbols Glossary at <https://www.dentsplysirona.com/ifu>.

© Dentsply Sirona 2026-03-26

[These Instructions for Use are based on Master Version 03]

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Dentsply DeTrey GmbH